

Indications of Clean or Replace Air Filter Foam

JAY series

There are two air filters in JAY serial models, first filter and the second one. Please see the following diagram1 and diagram4 for the two air filters.

It is recommended that the first filter should be cleaned or replaced every 100hours, the second filters shall be cleaned or replaced every 500hours .

I First filter

1. See Diagram 1 for the first filter



Diagram 1

Filters can be taken out and washed or replaced every 100 hours.

II Second filter

1. Open the top of storage box and take off the storage box by loosening the screw.



III Second filter

1. Find the second filter in in Diagram 4, there is an airway pipe connected to the third filter

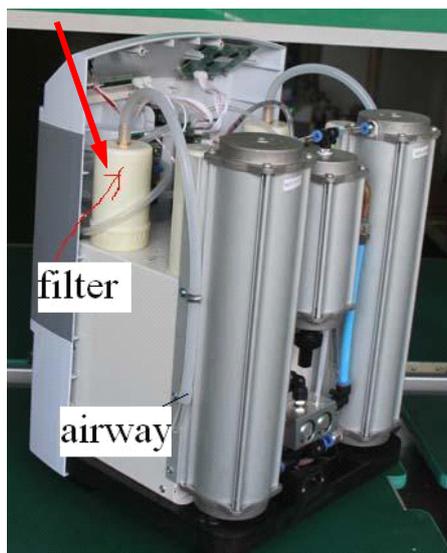


Diagram 4

2. Pull off the airway tube from the air filter, then loose the filter chamber in counter clockwise direction, then you may find green foam as in diagram 5 & 6.



Diagram 5



Diagram 6

3. Please check whether the green filter is become dark, if so, please clean it; If it becomes deep dark, please replaced it. This maintenance should be taken for every 500-800 hours. In the meanwhile, please see the filter is the round column as the right one of Diagram 5, if it is the same, it is OK. But if the top side of the filter is sealed, please cut it as diagram 6.



Diagram 7

4. Finally set back the filter foam to the filter column and tight it as Diagram 7 and diagram 4.

▲ **Damage:** Do not operate the concentrator without the filters installed, or while filter foam are wet. These actions could permanently damage the concentrator.